

The Milford Person Lift & Mobile Base



Key Features

- The aluminium Milford body weighs less than 8kg (17.5 lbs)
- Lifts people up to 111kg (17.5 stone)
- Fits all 2 & 3 door and most 4 or 5 door vehicles including estates, people carriers and 4x4's
- Lifts you onto your standard car seat
- Emergency up and down buttons
- Shaped extension arm fits around person
- Power is supplied by the vehicles 12 volt battery

The Milford Person Lift -
safety, comfort and
versatility from one lift.

Lifting into vehicles

The Milford Person Lift is designed to lift a person, safely and without putting strain on the carer/operator or person being lifted. Manufactured in lightweight aluminium the body of the lift weighs only 8kgs, whilst retaining great strength.

Cars – Even the smallest cars can be fitted with the Milford Person Lift (three door cars normally have larger doors than four/five door cars).

Key Benefits

- Can be used on the passenger side of all vehicles. For driver's side, some vehicles may be suitable
- Can be used on back seats in most people carriers
- The hoist comes complete with sling, electrics, mounting post, plates and steel for fixing the mounting post into the vehicle (with full fitting and operating instructions)
- It is guaranteed for 12 months against faulty parts or workmanship
- The Milford Person Lift lifts up and down. Sideways movements into or out of the car are done manually but are very easy due to the substantial pivot points on the lifting arm and extra bearing
- Heavier people can be lifted with a modified hoist – please consult Autochair Ltd
- The length of the lifting arm can be adjusted to suit all cars and people being lifted
- The sling can be fitted around the persons back whilst still seated in their wheelchair



1

Fit the sling down your back and under legs then connect



2

Using the hand control, lift person so that their feet fit over the cars sill



3

Manoeuvre the person into the vehicle



4

Using the hand control lower the person onto vehicle seat



5


Remove the extension arm and main body of lift prior to driving away

Milford Mobile Base

The mobile base gives you the freedom and independence not only in your own home but enables you to take your Milford Person Lift anywhere you'd like, friends houses, on holiday, or even in a caravan.

Key Benefits

- The base is taken apart into 4 easily manageable pieces, each weighing less than 5kgs and can be assembled in approximately three minutes. Power pack and charger are included
- The Milford can easily be carried in your vehicle and 25+ lifts can be obtained from a full battery

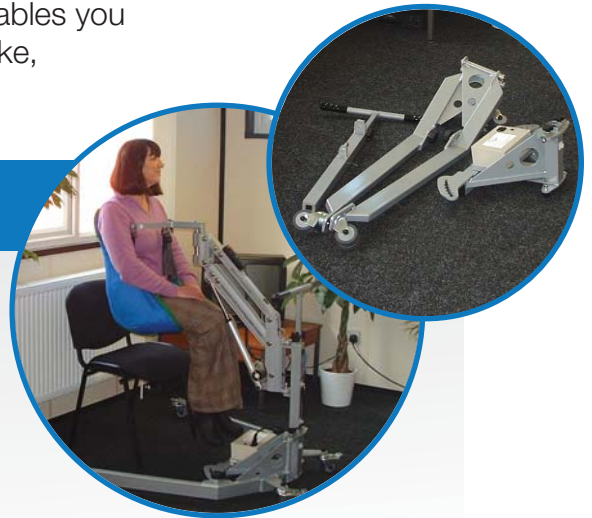


Wall Bracket

For confined areas where a mobile base is not suitable, the wall bracket comes into its own.



Floor Mounted


Mounted into the floor by a floor bracket that the Milford Person Lift fits onto, provides you with endless opportunities to travel without discomfort, hassle or worry for you or your partner.

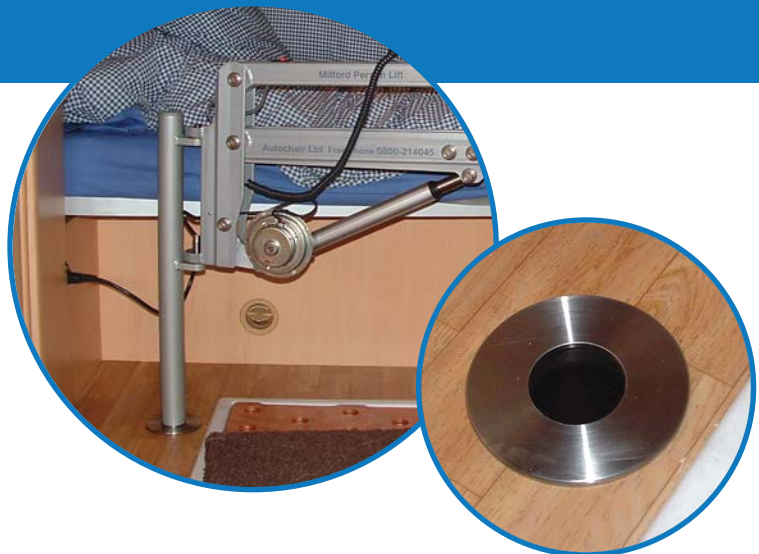

Lift Accessories



Mounting post
in car, lift removed

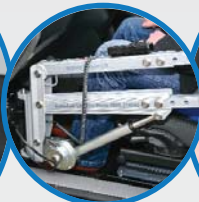

Lifting arm



Easy to
attach fixings



4 compartment
padded bag, with
carrying straps



Simple hand
control



5 standard
sling sizes
available

“I can’t thank Autochair enough for providing my mum with such a life changing product. We had the Milford fitted just before the New Year and we are currently driving to Inverness for a holiday and my mum is truly loving the experience, she can now look out of the window properly - the Milford has really changed all of our lives. Thank you Autochair for a wonderful and brilliant service and cracking product”

Fiona Crozier,
Somerset

Technical Information

- Lifting arm weight: 8kgs
- Maximum height range of lift: 23 inches
- Fuse rating: 25 amp
- Power supply: 12v
- Slings available: Extra small - Extra large including amputee sling
- Maximum user weight: 111 kgs (heavy duty option available)
- CE Marked



Motability accredited.
NMEDA UK member.
European Mobility
Group member.

Autochair have over 30 years experience in the design and manufacture of vehicle adaptations to assist people with disabilities and reduced mobility - **call us today to find your new freedom with Autochair: 0800 214 045**

- Wheelchair Hoists
- Scooter Hoists
- Swivel Seats
- People Lifts
- Wheelchair Carriers
- **SMARTDRIVING CONTROLS**

Autochair reserve the right to change product specifications or appearance without prior notice through product development and improvement. We aim to ensure our customers are aware of this at the time of ordering.